THERAPEUTIC SHOES

INSTRUCTIONS FOR USE

NL – GEBRUIKSAANWIJZING – THERAPIESCHOENEN
FR – INSTRUCTIONS D'UTILISATION – CHAUSSURES THÉRAPEUTIQUES
DE – THERAPIESCHUHE – GEBRAUCHSANWEISUNG
IT – LE SCARPE TERAPEUTICHE – ISTRUZIONI PER L'USO

ENGLISH

S.4

NEDERLANDS

S.16

FRANÇAIS

S.28

DEUTSCH

S.40

L'ITALIANO

S.52

For visually impaired, this document is available in PDF format with this QR code.

CONTENTS

Introduction	6
Product description	7
Purpose/Indications	7
Potential risks of use and contraindications	8
Directions for use	9
Safety instructions and residual risks	10
Cleaning and care	11
General rules	12
Technical specifications/Parameters	13
Disposal	13
Guarantee	14

INTRODUCTION

Dear clients,

Congratulations on the purchase of your new Perpedes therapeutic shoe. You have a high quality product selected which one with the utmost care manufactured, tested and packaged.

Please carefully read the instructions for use before using this product. Follow all instructions, particularly the safety instructions.

Perpedes GmbH
Härtwasen 8-14
73252 Lenningen
Germany

T. +49 (0)7021 738 30-0
info@perpedes.de
www.perpedes.com

PRODUCT DESCRIPTION

Perpedes therapeutic shoes can be used for various purposes and seasons. The orthotic shoes (OT) are indicated if, due to symptoms of an illness, a leg orthotic with a foot section and/or foot orthotics must be worn and if these orthotics cannot fit into regular mass-produced shoes. For leg orthotics with different volumes, orthotic shoes are offered with a 3-width system. Anti-varus shoes (AV) are used, with support from orthopedic insoles if necessary, for the correction of children's pigeon toes, Pes adductus and post-operative clubfoot. If an orthotic or anti-varus shoe is worn on one foot, then the opposite side is freely selectable in width and size.

PURPOSE/INDICATIONS

Orthotic shoes (OT) are for patients who rely on wearing an AFO or DAFO and/or foot orthotics because of symptoms of an illness. The orthotic shoes have enough space to fit in orthotics. If orthotic shoes are only worn on one foot the opposite side is freely selectable in width and size. Anti-varus shoes (AV) are used for the treatment and correction of children's pigeon toes, Pes adductus and post-operative clubfoot, with support from orthopedic insoles if necessary. Neutral shoes (NT) are used to balance out anti-varus shoes if they are only worn on one foot.

POTENTIAL RISKS OF USE AND CONTRAINDICATIONS

In the following cases, you should consult your doctor before using the product:

- Skin conditions or injuries in the area where the shoe is worn
- Indications of inflammation, such as reddening, warming or swelling
- Circulatory disorders
- Lymphatic drainage disorders
- Sensory disorders in the area where the shoe is worn

If you should notice any unusual changes, seek the advice of a doctor immediately.

DIRECTIONS FOR USE

Before the therapeutic shoes are used for the first time, a trained specialist must ensure that the shoes are adjusted to the needs of the user. At the time of delivery, the specialist is responsible for the correct fit and function of the shoes. If this is not done properly, the product liability is not guaranteed. The correct method to tighten the shoes is shown below:

1. Open the shoe closure so that it is open wide enough for the foot and fold the tongue forwards.
2. Check the shoe for foreign objects and remove them before entering the shoe.
3. Place the foot into the opened shoe, with the AFO/DAFO fastened on if necessary.
4. Take care to ensure that the foot is placed securely/ correctly in the shoe. This means the heel should be as far back in the shoe as possible.
5. Depending on the model, close the shoe with the Velcro straps, zip, BOA closure or laces. The foot, or the foot with an AFO or DAFO or with the foot orthotic, must be well secured. The shoe should not be too loose or too tight.
6. The Velcro straps can be individually shortened depending on the model.

SAFETY INSTRUCTIONS AND RESIDUAL RISKS

The adjustment, finishing and instructions for use of the therapeutic shoes should only be performed by trained personnel.

The product is intended for use by one patient only.

Should pain, swelling, numbness or changes in sensitivity, unusual reactions or an uncomfortable feeling occur whilst wearing the shoes, immediately contact your doctor or orthopedic shoe technician.

The product must not be used in water and must be protected against temperatures over 50°C. Direct UV exposure (e.g. sunlight) can alter the color or the therapeutic shoes. This change has no detrimental effect on the function of the shoes.

If the product gets damaged, please replace it immediately or get it repaired by a qualified specialist.

In certain countries, Perpedes therapeutic shoes can be prescribed by a doctor. The shoes should only be used according to information in these instructions and within the scope of their intended purpose.

At icy surfaces or with wet outsoles, there may be an increased risk of slipping.

Please be particularly attentive at first use and get used to the product on even and paved surfaces.

Please make sure the product is stored away from moisture.

CLEANING AND CARE

To extend the life of the product, any dirt should be removed with a soft, and if necessary damp, brush after use.

Do not use any cleaning products or solvents.

The shoes should never be cleaned in a washing machine. Dry the shoes at room temperature in a well-ventilated area, away from direct UV exposure (e.g. sunlight) and without using an external source of heat.

Sand and seawater can damage the therapeutic shoes. Clean the shoes thoroughly if they have been subjected to such conditions.

LEATHER

Leather is a natural product. To ensure the leather's functionalities, we recommend that you treat it with normal leather care products.

Please note that the treatment can alter the appearance of the shoes. This does not affect the function and characteristics of the shoes. Please test cleaning supplies first in an inconspicuous place.

INNER LININGS

Socks should be worn and it is recommended to clean the lining and inside the shoe regularly.

LACES

Laces and other fasteners are everyday objects which can wear out. They should be replaced when necessary to ensure that product continues to function well.

VELCRO FASTENERS

The function of the Velcro fasteners is only guaranteed if they are kept clean. Clean as required.

GENERAL RULES

- Due to the selection based on indication, individual adjustment and on hygiene grounds, a therapeutic shoe should only be used by one patient. A therapeutic shoe is not suitable for multiple use (re-use).
- Check the product for wear and damage regularly, but latest before each use. The functional characteristics of the therapeutic shoes may decrease due to repeated use (wear caused by being worn). Therefore, it is not possible to determine a definitive life span or wearing period. The shoes should be replaced when they wear out.
- Therapeutic shoes are intended to be used with other medical products. However, they may only be combined with orthopedic appliances when they have been selected and adjusted in compliance with the indications by a trained specialist.

- Please note, leather is a natural product and in addition to color differences may also have defects in the skin structure (e.g. scars), which do not entitle to a complaint about the functioning product. These features are the peculiarities of a natural material and should not be seen as a defect.

TECHNICAL SPECIFICATIONS/PARAMETERS

The material of the therapeutic shoes can vary depending on the model:

- Leather
- Textile fabric
- Lambskin
- EVA (ethylene vinyl acetate)
- TPU (thermoplastic polyurethane elastomer)
- Rubber
- Polyamide (Velcro fasteners)
- Metal (eyelets, zips)

DISPOSAL

The product and the packaging must be disposed of according to the legal guidelines in the country in which the shoes are used.

GUARANTEE

The currently applicable legal regulations apply. Non-observance of the directions for use, improper cleaning, as well as independent technical modifications and additions to the product will lead to the expiration of the guarantee and product liability.

The CE marking confirms compliance with the requirements of Regulation (EU) 2017/745 on medical devices (MDR).

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